

Определение активности фактора VIII (FVIII REN)

Приготовление реагентов проводить в соответствии с [инструкцией](#) к набору.

Путь написания протокола

[Menu] → [Setup] → [Test Protocol] → [_____]

[page 1]

Identification										
Abbreviation		FVIIIREN				Method		clot-based		
Name		FACTOR VIII ACTIVITY								
Sample			Diluant							
Volume	Incu.	Dil.	ID	Name	Vial	Stab.	Min.Volume			
50 µl	0 sec	1/10	11361	OWREN-KOLLER	15 ml	72 h	0.90 ml			
Reagents										
ID	Name			Incub	Vol.	Vial	Stab	Min. Volume	Washing	
			sec	µl	ml	h	(ml)	Before	After	
Ra 75	PLSUBVIII			0	50	10	168	0.10	no	normal
Rb 76	APTREN			180	50	10	168	0.10	no	normal
Rc										
Rd 77	CACLREN				50	10	168	0.10	no	normal
Analysis				Results				Validation (%)		
Min. Time		5		Primary Units		%		Min.		Max.
Max. Time		180		Corrector		1.000		5.00		300.0
Mean. Time		50		Single/Duplicate		single		Redil. Condition (%)		
Rd Heating		Yes		Precision		5.00 %		1/5 < 50		
Stirring		no						1/20 > 150		
Clot Type		normal								

[page 2]



C A L I B R A T I O N									
MODE Graph - Linear regression							Scale Raw Data Lin Concentration Log Single / Duplicate single		
Calibration				Vial	Stab	Min. Volume			
ID	Key	Name	Dil.	ml	h	ml			
51		CALPLASMAREN	1/5	1	4	0.1			
51		CALPLASMAREN	1/10	1	4	0.1			
51		CALPLASMAREN	1/20	1	4	0.1			
51		CALPLASMAREN	1/40	1	4	0.1			
51		CALPLASMAREN	1/80	1	4	0.1			
51		CALPLASMAREN	1/160	1	4	0.1			
							Offset corrector		
							ID		
							Key		
							Name		
							Vial	Stab.	Min. Volume
							ml	h	
Controls				ID	Key	Name	Vial	Stab.	Min. Volume
							ml	h	(ml)
Level 1				52	1	PLASMA N	1	8	0.1
Level 2				53	1	PLASMA P	1	8	0.1

[page 3]

Printout / Transmission					Usual Values (%)			
	Unit	Convers. Factor	Print.	Transmission Test Number	Min.	Max.		
Main	%		Yes	0	1.00	150.00		
Aux. 1	sec.		Yes	0				
Aux. 2								
Aux. 3								
					Printout Limits			
					Min.	1.00 %		
					Max.	300.00		
Quality Control					Period	Vial	Stab.	Min. Volume
	ID	Key	Name		h	ml	h	(ml)
Level 1	52	1	PLASMA N		24	1	8	0.1
Level 2	53	1	PLASMA P		24	1	8	0.1
Level 3								

После проверки правильности протокола выйти из [Setup] с помощью [Esc]→[Save and Exit]. Для подтверждения изменения протокола ввести пароль: [CQ] или [QX].